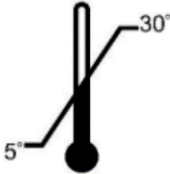


PRODUCT DATA PACK (PDP)	PDP REF: PDP-B3015F- ANIGENE WIPE V03	
NAME: ANIGENE Professional Surface Disinfectant Wipes CODE: B3015F		

PRODUCT INFORMATION	
PRODUCT NAME	ANIGENE Professional Surface Disinfectant Wipes
PRODUCT CODE	SKU100A
FORMULATION VARIANTS	B3015F – Citrus
NOTE: ACTIVE INGREDIENTS AND SAFETY INFORMATION DETAILED IN SDS	
DIRECTIONS FOR USE	Suitable for use on hard, non-porous surfaces. Lift seal and remove one wipe from pack. Place palm of hand on top of the wipe. Apply the wipe to the surface in an 'S' pattern, ensuring full coverage. Each wipe is suitable for disinfecting 1m². Avoid wiping the same area twice and use a new wipe for each new surface, to avoid cross-contamination. Leave surface to dry naturally, no need to rinse. Test an inconspicuous area before use if unsure. Reseal pack after use to avoid wipes drying out. Always rinse food / water bowls and toys after disinfection. Dispose of used wipes in general
APPLICATION AREAS	PT 3 – Veterinary Hygiene
STORAGE CONDITIONS	Keep out of reach of children. If medical advice is needed, have product container or label at hand. Avoid release to the environment. Dispose of contents/container in accordance with national regulations. Incompatibility with other materials: Oxidising Substances, Anionics, Hypochlorite, Aldehydes. 
SHELF-LIFE	2-Years Established following criteria for accelerated storage test in section 3.4.1.1 of ECHA guidance on the BPR: Volume I. Part A. Chapter III: Requirements for Biocidal Products. Version 1.1 November 2014
If printed check that the PDP is the most up to date version according to the website	

PRODUCT SPECIFICATION	
APPEARANCE	Colourless to pale yellow impregnated wipes
COLOUR	Colourless to pale yellow
ODOUR	Characteristic
pH	9.00 – 10.00
SPECIFIC GRAVITY	0.990 – 1.020
VISCOSITY	N/A

PRODUCT DATA PACK (PDP)	PDP REF: PDP-B3015F- ANIGENE WIPE V03	
NAME: ANIGENE Professional Surface Disinfectant Wipes CODE: B3015F		

BACTERICIDAL EFFICACY					
TEST REF	TEST ORGANISMS	TEST CONDITIONS	TEST RESULT	REPORT REF	COMMENTS
EN1656	<i>Enterococcus hirae</i>	1 min, low soil, 20°C	>5 log reduction	TR-910	None
EN16615	<i>Enterococcus hirae</i>	1 min, low soil, 20°C	>5 log reduction	TR-913	Modified stainless steel
EN1656	<i>Pseudomonas aeruginosa</i>	1 min, low soil, 20°C	>5 log reduction	TR-910	None
EN16615	<i>Pseudomonas aeruginosa</i>	1 min, low soil, 20°C	>5 log reduction	TR-913	Modified stainless steel
EN1656	<i>Staphylococcus aureus</i>	1 min, low soil, 20°C	>5 log reduction	TR-910	None
EN16615	<i>Staphylococcus aureus</i>	1 min, low soil, 20°C	>5 log reduction	TR-913	Modified stainless steel
EN1656	<i>Proteus hauseri</i>	1 min, low soil, 20°C	>5 log reduction	TR-911	None
EN16615	<i>Proteus hauseri</i>	1 min, low soil, 20°C	>5 log reduction	TR-913	Modified stainless steel
EN1656	<i>Escherichia coli</i>	1 min, low soil, 20°C	>5 log reduction	TR-911	None
EN1656	<i>Brucella intermedia</i>	1 min, low soil, 20°C	>5 log reduction	TR-926	None
EN1656	<i>Bordetella bronchiseptica</i>	1 min, low soil, 20°C	>5 log reduction	TR-926	None
EN1656	<i>Streptococcus equi</i>	1 min, low soil, 20°C	>5 log reduction	TR-926	None
EN1656	<i>Salmonella typhimurium</i>	1 min, low soil, 20°C	>5 log reduction	TR-927	None
EN1656	MRSE (<i>Staphylococcus epidermidis</i>)	1 min, low soil, 20°C	>5 log reduction	TR-927	None
EN1656	MRSP (<i>Staphylococcus pseudintermedius</i>)	1 min, low soil, 20°C	>5 log reduction	TR-927	None

YEASTICIDAL/FUNGICIDAL EFFICACY					
TEST REF	TEST ORGANISMS	TEST CONDITIONS	TEST RESULT	REPORT REF	COMMENTS
EN1657	<i>Candida albicans</i>	1 min, low soil, 20°C	>4 log reduction	TR-912	None
EN16615	<i>Candida albicans</i>	1 min, low soil, 20°C	>4 log reduction	TR-914	Modified stainless steel
EN1657	<i>Aspergillus brasiliensis</i>	30 min, Low soil, 20°C	>4 log reduction	TR-921 v02	None
EN16438	<i>Aspergillus brasiliensis</i>	30 min, Low soil, 20°C	>3 log reduction	TR-922	None
EN16615	<i>Aspergillus brasiliensis</i>	10 min, Low soil, 20°C	>4 log reduction	TR-987	Modified for fungi on stainless steel

PRODUCT DATA PACK (PDP)	PDP REF: PDP-B3015F- ANIGENE WIPE V03	
NAME: ANIGENE Professional Surface Disinfectant Wipes CODE: B3015F		

EN1657	<i>Trichophyton mentagrophytes</i>	10 min, Low soil, 20°C	>4 log reduction	TR-988	None
EN16438	<i>Trichophyton mentagrophytes</i>	30 min, Low soil, 20°C	>3 log reduction	TR-989	None
EN16615	<i>Trichophyton mentagrophytes</i>	10 min, Low soil, 20°C	>4 log reduction	TR-990	Modified for fungi on stainless steel

VIRUCIDAL EFFICACY					
TEST REF	TEST ORGANISMS	TEST CONDITIONS	TEST RESULT	REPORT REF	COMMENTS
EN14675	Vaccinia virus	5 min, low soil, 20°C	>4 log reduction	TR-917	None
EN17122	Feline coronavirus	5 min, low soil, 20°C	>4 log reduction	TR-918	None

ADDITIONAL DATA		
TEST METHOD	RESULT	TEST REPORT REF
Preservative Efficacy Testing	B3015 complies with the test for the Efficacy of Antimicrobial Preservatives (European Pharmacopoeia 11 th Edition)	TRC-338