

# Alcohol-free Surface Disinfectant Wipes

60sec

2in1

Apple

x200

ⓘ Always Read Product Label and Material Safety Data Sheet before use. For Professional use only.

- Specially formulated and effective against all enveloped viruses, including Coronavirus, HIV, Hepatitis, Influenza and many other microorganisms. (for full details, see overleaf)
- For surfaces and non-invasive medical devices. Ideal to use on surfaces sensitive to alcohol.
- Contains detergent derived from renewable sources.
- Wipes have an apple aroma and are supplied in packs of 200 wipes.
- Each wipe measures 200 x 200mm.
- 2 in 1 product, wipes contain detergent for added cleaning power.

[g9chemicals.com](http://g9chemicals.com)



Your only solution

# Alcohol-free Surface Disinfectant Wipes

60sec

2in1

Apple

x200

! Always Read Product Label and Material Safety Data Sheet before use. For Professional use only.

## Contact Times

Bactericidal – 60 sec  
Yeasticidal – 60 sec  
Enveloped Viruses – 60 sec

## Disinfectant Test Portfolio

The following test standards have been achieved: EN14476, EN13727, EN13624, EN16615, EN14885

### Bactericidal:

Pseudomonas aeruginosa,  
Enterococcus hirae,  
Staphylococcus aureus

### Yeasticidal:

Candida albicans

### Enveloped Viruses:

Poxviridae, Herpesviridae,  
Filoviridae (e.g. Ebola, Marburg),  
Hepatitis B virus (HBV), Hepatitis  
C virus (HCV), Hepatitis Delta virus  
(HDV), Influenza virus, Rubella  
virus, Measles virus, Rabies virus,  
Coronavirus (e.g. SARS, MERS),  
Human immunodeficiency virus  
(HIV) & Human T cell leukemia virus  
(HTLV), Vaccina Virus

## Directions For Use:

Using a new wipe for each application, wipe objects to be disinfected so that they are thoroughly moistened and let the disinfectant take action over a minimum of 60 seconds (to achieve full efficacy, leave for five minutes). Allow surface to dry or wipe with clean tissue.

[g9chemicals.com](http://g9chemicals.com)



G9 Chemicals Ltd, 24 High St,  
Charing, Ashford, Kent,  
TN27 0HX, UK

+44 (0)1580 895 141  
[info@g9chemicals.co.uk](mailto:info@g9chemicals.co.uk)  
[g9chemicals.com](http://g9chemicals.com)